


SkyJet® protocol of use with XL Stabi ZO®

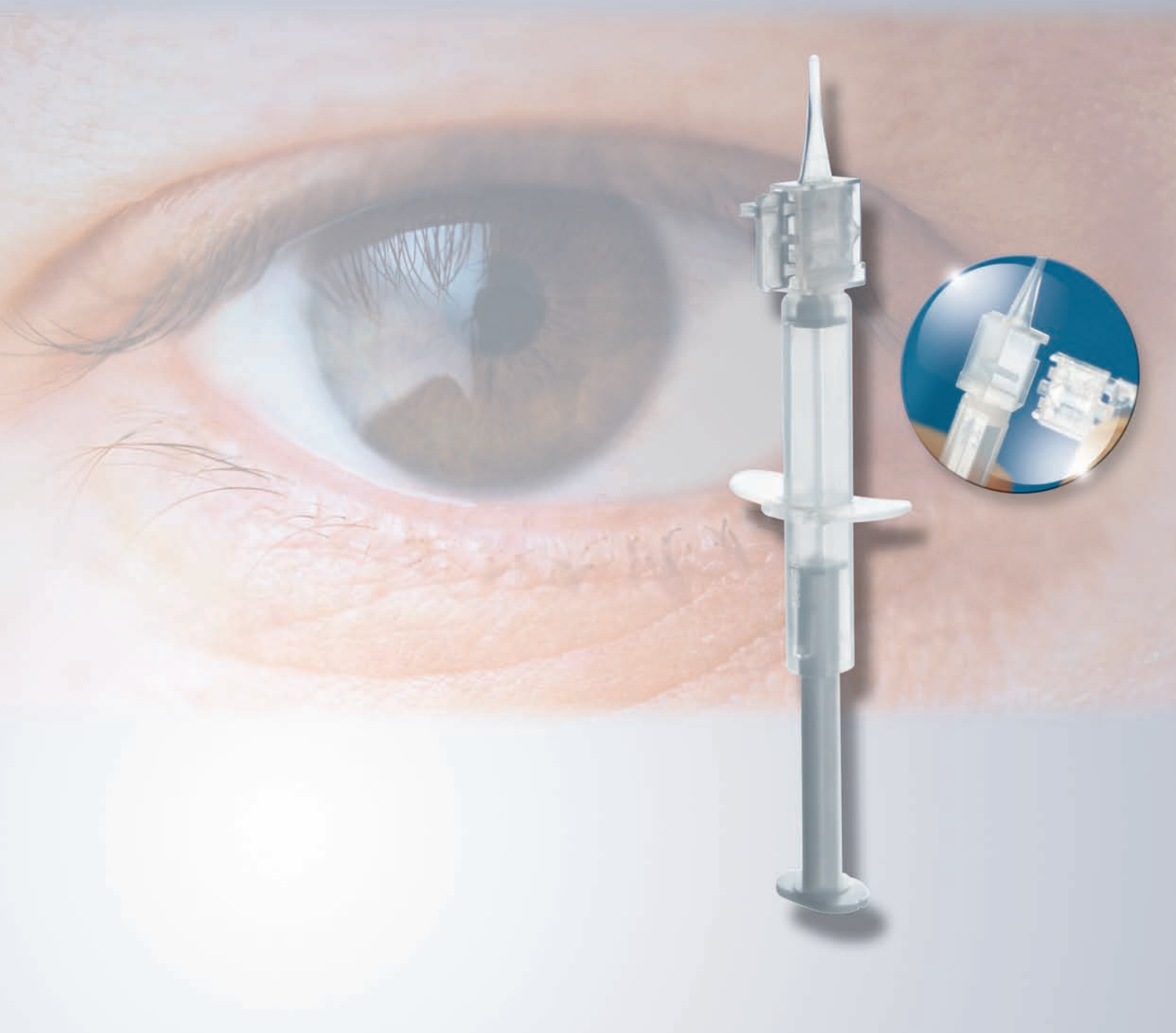

SkyJet® protocol of use with XL Stabi ZO®



Subject to change in design and scope of delivery and as a result of ongoing technical development. SkyJet® and XL Stabi ZO® are registered trademarks of Ioltech.



Check that the plunger is fully withdrawn and held by the safety clips.



Insert the cassette into the injector following the arrows. Push all the way in until you hear a "click".

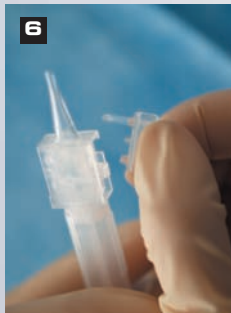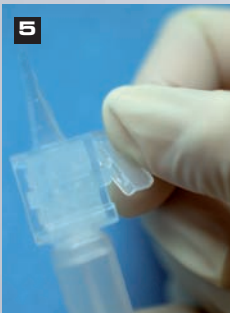

Hold off the breakable safety tab between thumb and index finger. Snap it in the clockwise direction. The tab breaks and releases the lens.



Introduce viscoelastic into the hole on the back of the injector. Fill the first 3 mm of the injector nozzle up to the line.



Gently arm the plunger to release it from its safety clips and slowly push to drive the lens into the injector nozzle.



With the bevelled orifice facing downwards, insert the tip of the injector into the incision and inject the lens into the capsular bag.

Carl Zeiss Meditec SAS
Avenue Paul Langevin - BP 5
17053 La Rochelle Cedex 9
France

Tel.: +33 (0) 5 46 44 85 50
Fax: +33 (0) 5 46 44 85 60
E-mail: czmlr.contact@meditec.zeiss.com
www.meditec.zeiss.com/iol

